

emexact[®]


Instructions for import of the implant library
EM Exact into DentalWings software



How to import the EM Exact Library


Import the library into the DWClient

1. Select *Applications* → *Implant Kits*

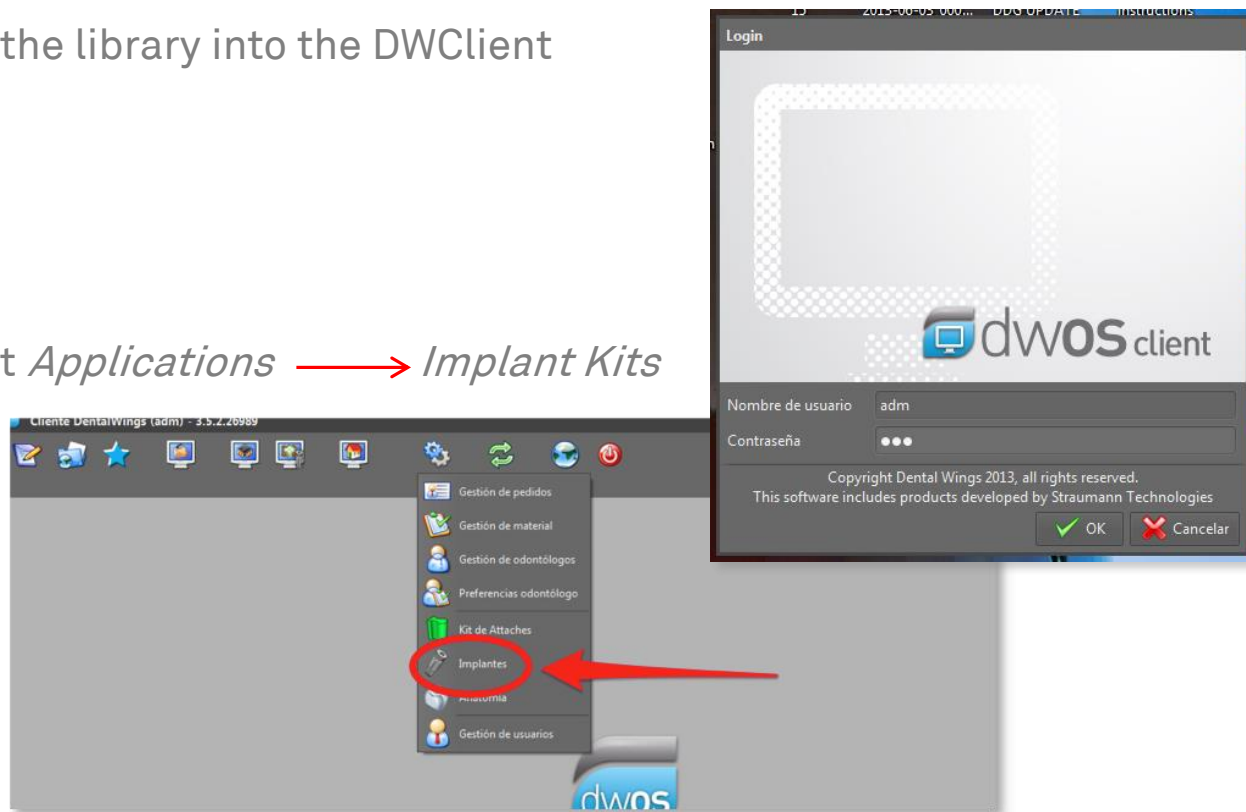


How to import the EM Exact Library

2. Right click on *Implant Kits* and choose *Import Implant Manufacturer Kits*

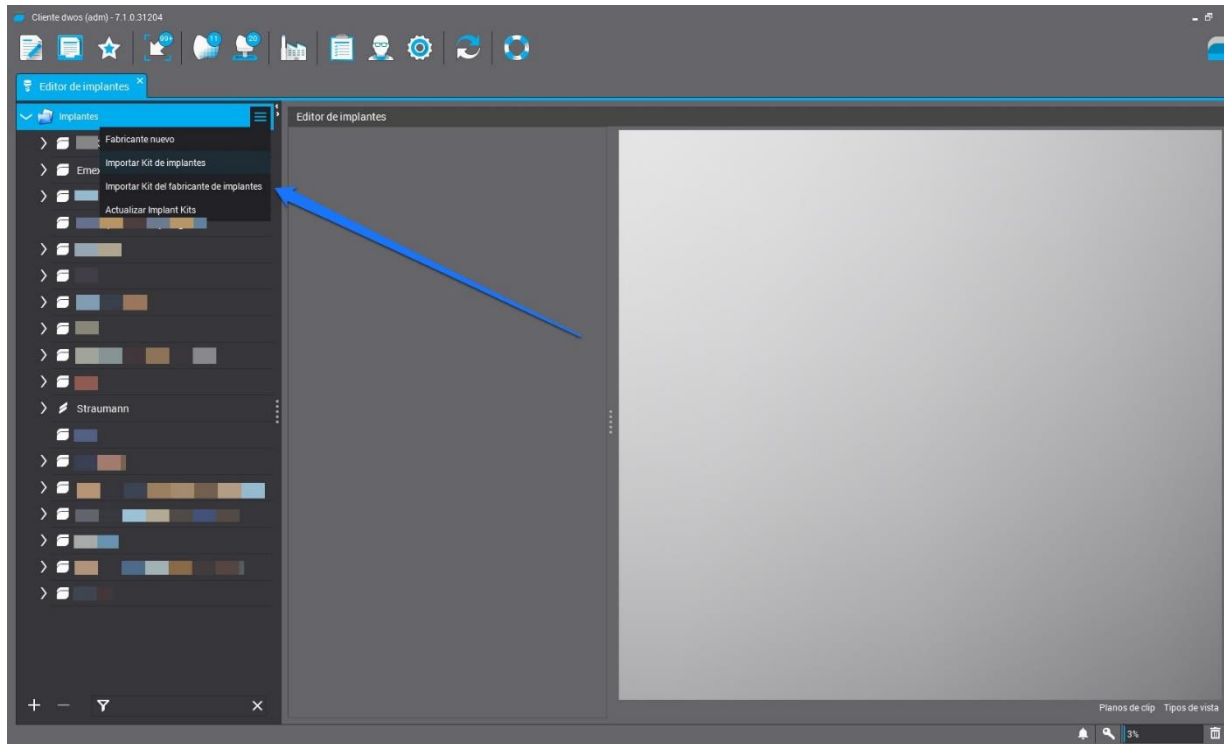


How to import the EM Exact Library

3. Identify the path of the EM Exact *Library* and choose *open*

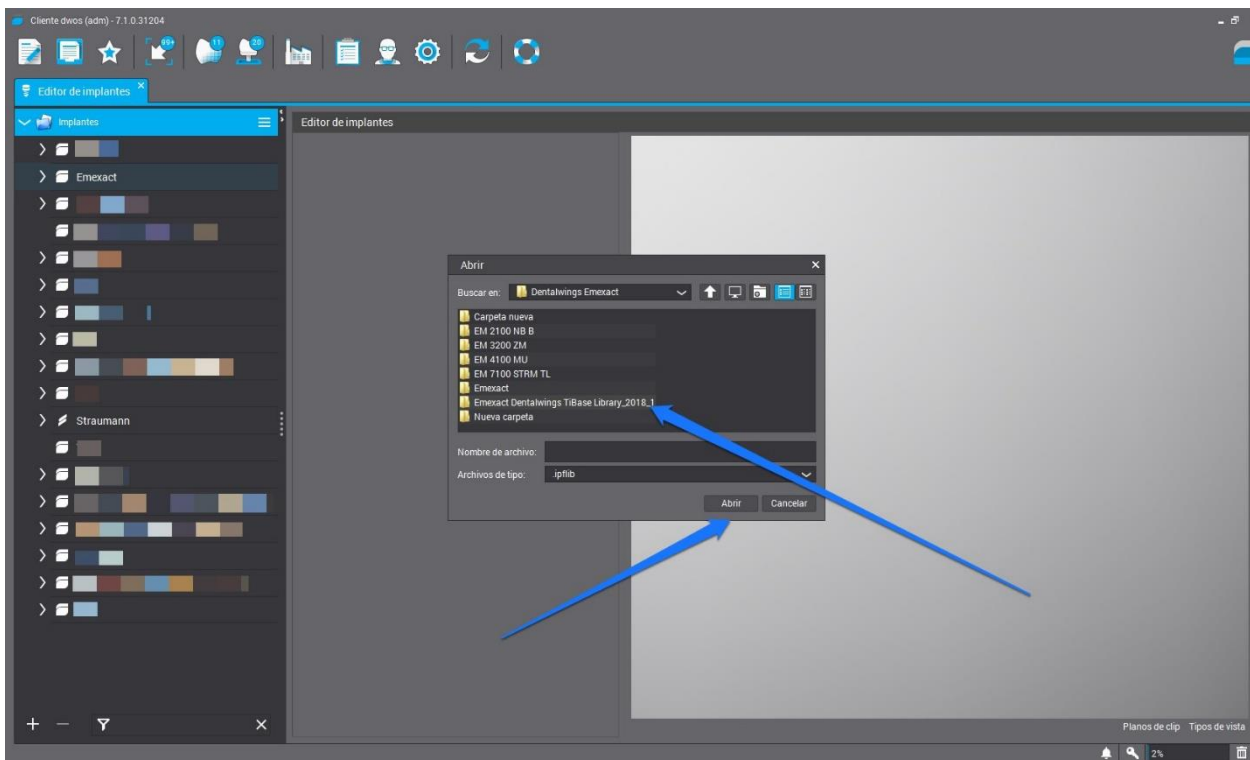


How to import the EM Exact Library

5. When the library is successfully imported, it will be added to the available implant Library list. All Kits of EM Exact Library will be integrated.

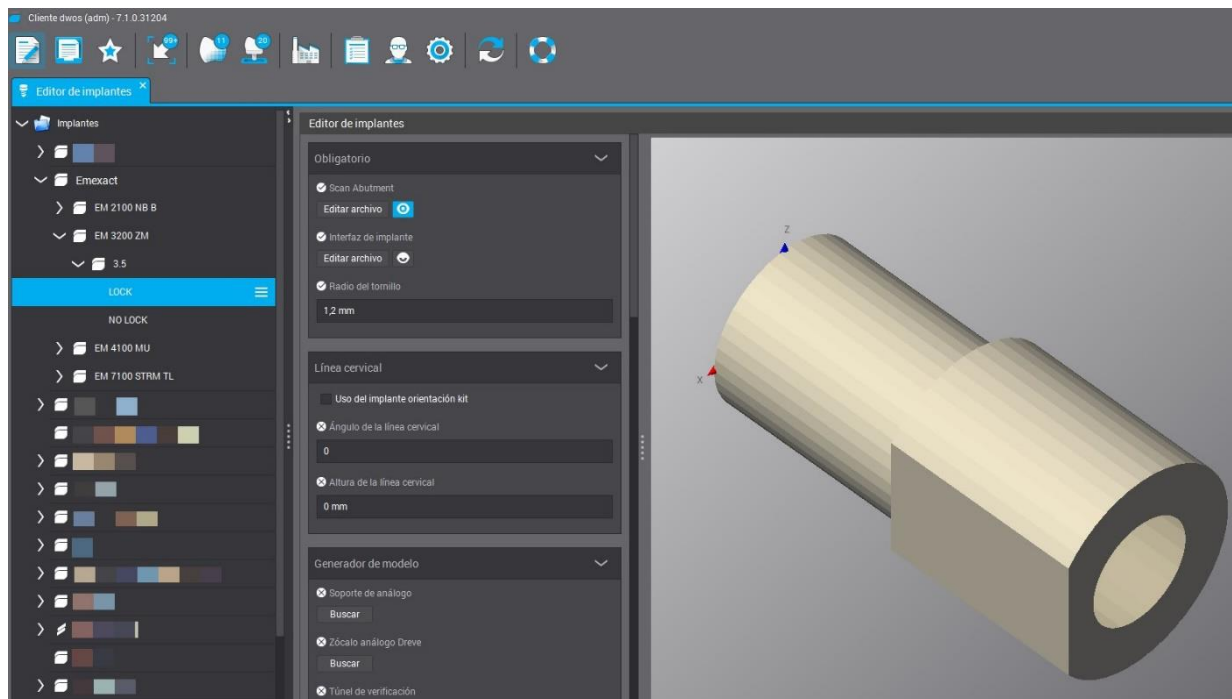


How to import the EM Exact Library

When you select a kit, displays a screen with all components of this Kit

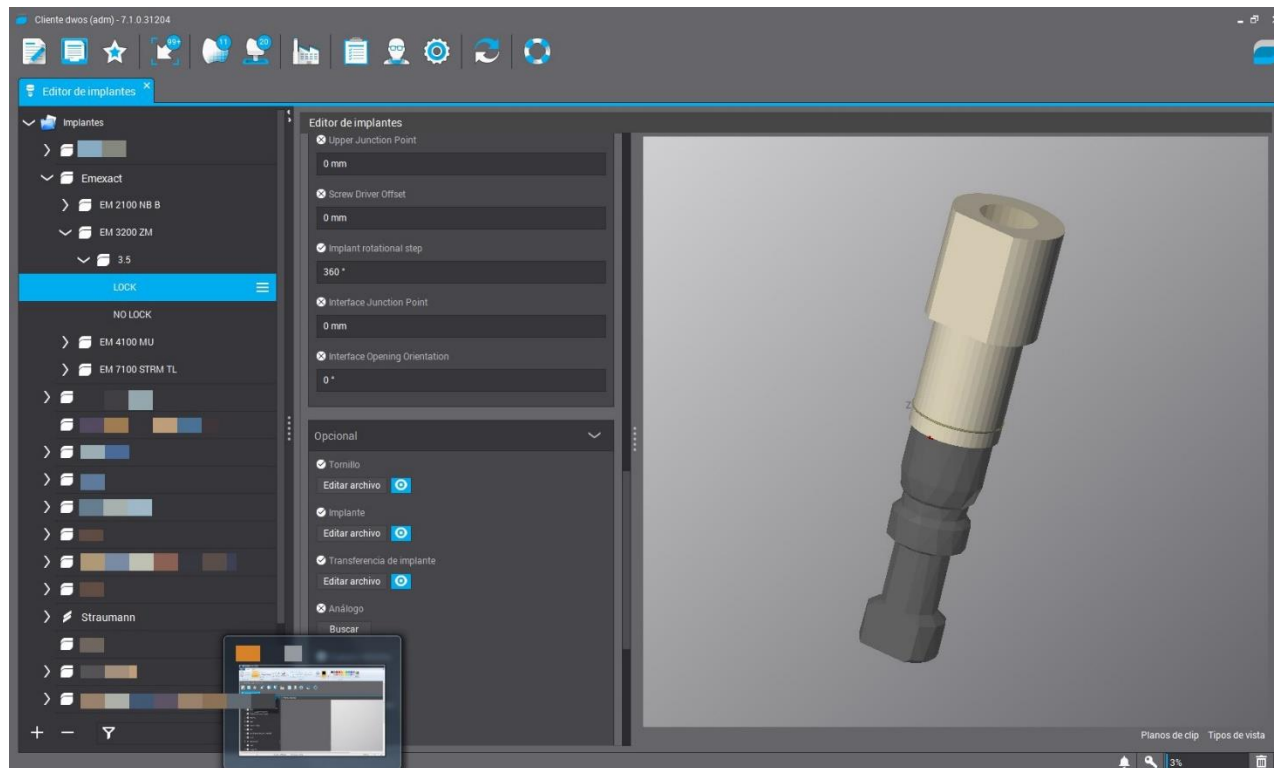


How to import the EM Exact Library

6. Now you can select the connection that you want to design

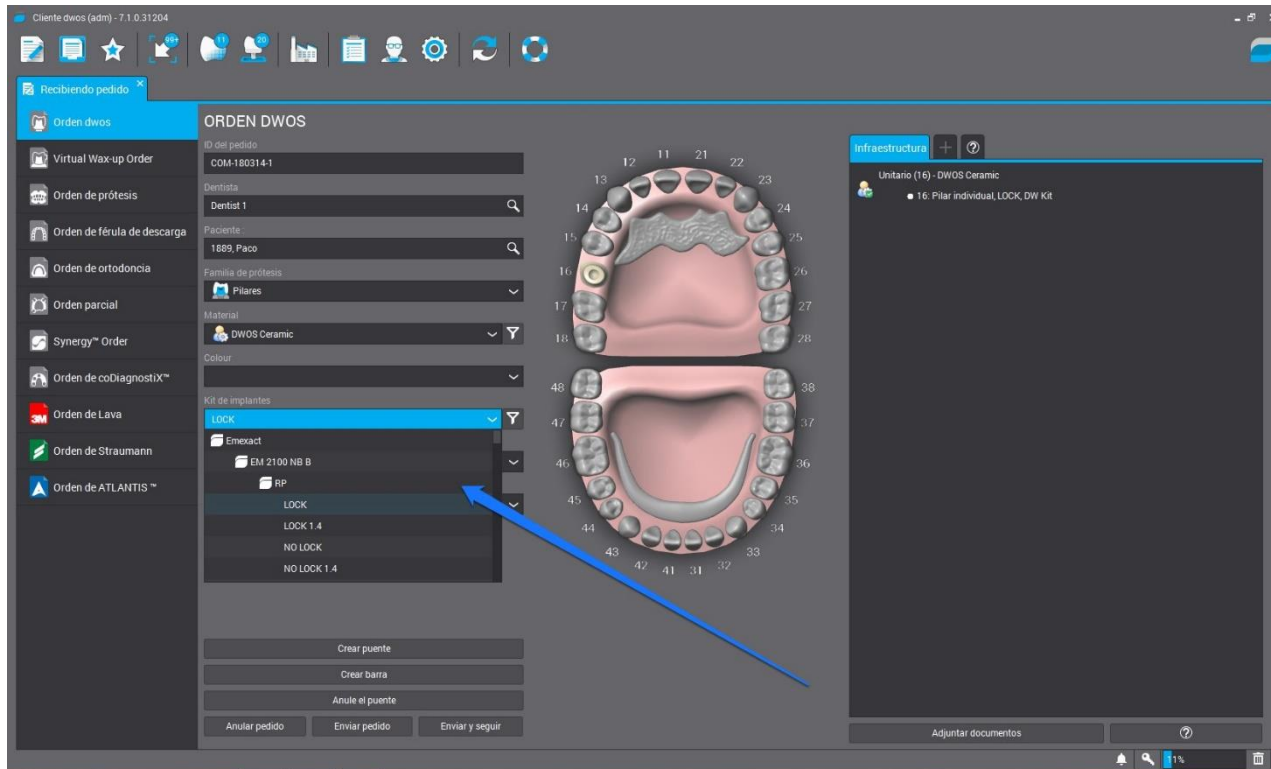

EM Exact Implant Library for DentalWings

Everything is ready to design this:

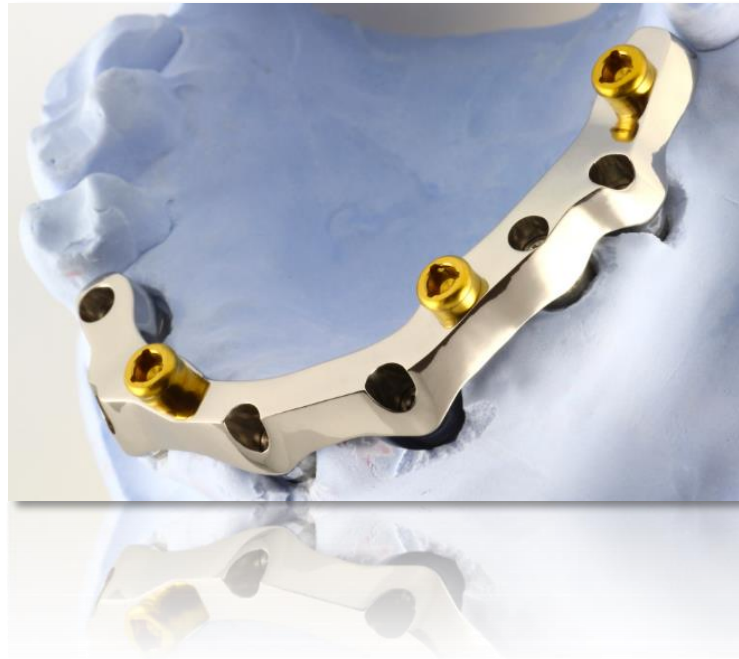

Thanks for Watching

EM Exact Support